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
Flamozil®

Paraben-free, moisture-regulating hydrogel.

For the treatment of local, dry or moderately moist wounds as well as superficial burns (1st and light 2nd degree)

The skin

The skin is the largest organ of the human body and is our daily protection against harmful influences from outside.

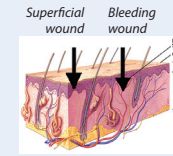When this protective layer is damaged, we are vulnerable for infections and inflammation. Although the body can restore skin by itself, the right type of wound care is necessary to prevent complications during the healing process, such as the formation of scar tissue.

Wound care

The body reacts to a wound by forming a scab as quickly as possible. Although this is the fastest way to seal the protective layer again, this scab finally slows down the healing and increases the chance of cicatrization. To ensure an optimal recovery of the skin, it is better to keep the wound slightly moist.

Flamozil® is a perfect solution to regulate the moisture content in the wound bed. In case of a dry wound, the Flamozil® formula releases water thus preventing the formation of scabs and skin cracks.

In case of a moist wound, the Flamozil® formula absorbs the excess wound fluid and in doing so reduces the chance of infection and improves the healing process.

Flamozil® also has a cooling and soothing effect and prevents the secondary bandage from sticking to the wound bed. This makes bandage changes less painful.

When to use?

- burns 1°-2° degree
- grazed skin
- cutting wounds
- open leg sores
- surgical wounds
- bedsores
- sun burn

Purpose

- faster wound healing
- sooth pain
- avoid infections
- prevent permanent skin damage
- reduce the formation of scar tissue

Instructions for use

Closed wounds:

- Clean and disinfect the wound.
- Apply Flamozil® without touching the wound with the spray can.
- Use a bandage or plaster if necessary.

Open wounds:

- Clean and disinfect the wound. Carefully pat dry.
- Apply a thick layer (0,5 - 0,75 cm) to the wound.
- Apply a bandage or plaster.
- Leave the bandage in place for one to three days until the gel has dissolved.
- Reduce the application of Flamozil® as a scab appears on the wound.