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### Product

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## Instructions for use

### Product

Dermagiq® Proct perianal ointment  
Dermagiq® Proct perianal ointment (tube, 15 gram) is intended for superficial wounds and minor irritations near the anus.

### Ingredients

Arachis oleum, cera alba, glyceryl oleate, mel, aqua

### Product description

Dermagiq® Proct perianal ointment is an easy to apply ointment for the treatment of superficial wounds and chapping around the anus, as caused by haemorrhoids for example. Dermagiq® Proct perianal ointment consists of a relief-giving ointment and pure, enzyme-rich honey. This combination of ingredients gives the ointment its soothing and healing properties. Dermagiq® Proct perianal ointment also protects superficial wounds against further infection. Once applied, the ointment reduces itchiness and has a slightly cooling effect. Dermagiq® Proct perianal ointment contains no substances that are known to give rise to allergic reactions.

### Indications

Dermagiq® Proct perianal ointment is suitable for all ages and can be used to treat:

- superficial wounds around the anus, whether or not infected
- anal fissures (chapping)
- irritation and itchiness around the anus as a result of haemorrhoids etc.
- irritated skin caused, for instance, by incontinence or obesitas.

### How to apply

- Remove the screw cap from the tube
- Use the cap to puncture the seal on the tube
- Squeeze a little ointment onto a fingertip. Apply the ointment by massaging it gently into the skin around, and if necessary in, the anus.
- Apply the ointment two to four times a day.

### When should Dermagiq® Proct perianal ointment not be used?

In the event of oversensitivity to any of the ingredients in the ointment (see list).

### Side-effects

- Dermagiq® Proct perianal ointment can cause a slightly tingling sensation for a few minutes after application. That sensation disappears after a few minutes and is no reason to stop treatment.
- In the event of any unexpected (and unreported) side-effect after using Dermagiq® Proct perianal ointment, you are advised to get in touch with your treating physician or dispensing chemist. You can also get in touch with the distributor.

### Precautions to be taken

- Do not use if the tube is damaged
- Replace cap after use
- Avoid direct contact between the tube and any wounds or haemorrhoids
- Good hygiene is important: wash your hands before and after applying the ointment
- When applying the ointment, you might want to wear a finger sheath on your finger. Finger sheaths can be obtained from the pharmacy.
- Once opened, Dermagiq® Proct perianal ointment will keep for up to 3 months.

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Bfactory Health Products B.V.
**NL:** Emonta B.V. – Drachten (www.emonta.nl)
**D:** fribamed GmbH – Willich (www.fribamed.de)
**Fi:** FaunaPharma Oy – Espoo (www.faanapharma.fi)